



VAH

IHO

ÖGHMP

Disinfection wipes

# DESCOCEPT AF WIPES

Alcoholic wipes for rapid disinfection

- Universal, comfortable and quick application using a practical dispenser
- Basic, rapid efficacy
- Especially suited for disinfection of small-sized surfaces in routine areas



Product description

DESCOCEPT AF WIPES are alcoholic, ready-to-use wipes for the disinfection and cleaning of alcohol-resistant medical devices and medical inventory, as well as surfaces of all kinds. DESCOCEPT AF WIPES offer a time-saving disinfection and cleaning application of small surfaces. The towels can be removed individually from the practical and reclosable dispenser box.

Range of application

According to Medical Device Directive (MDD): Rapid disinfection and cleaning of alcohol resistant, non invasive medical devices.

According to Biocidal Regulation (BPR):

For rapid disinfection and cleaning of alcohol-resistant medical inventory an all kinds of surfaces.

Application

Wipe surfaces with DESCOCEPT AF until completely wet and allow to dry for the entire contact time.

Application notes

Shelf life after opening: 3 months.

Spectrum of efficacy and contact times			30 s	1 min	2 min	5 min	30 min
<b>Bacteria and yeasts</b>							
application recommendation for surface disinfection: Bactericidal, yeasticidal*	VAH / EN 16615	with mechanical action, dirty conditions			•		
tuberculocidal (M. terrae)	EN 14348	clean conditions		•			
mycobactericidal (M. avium and M. terrae)	EN 14348	dirty and clean conditions			•		
<b>Viruses</b>							
active against enveloped viruses	RKI/DVV	incl HBV/HIV/HCV	•				
	EN 14476	dirty and clean conditions	•				
active against noroviruses (MNV)	EN 14476	clean conditions				•	
active against rotaviruses	RKI/DVV			•			
<b>Additional test results</b>							
bactericidal	EN 1040		•				
	EN 1276	clean conditions		•			
	EN 13727	clean conditions	•				
		dirty conditions		•			
EN 13697	clean conditions		•				
yeasticidal (Candida albicans)	EN 1275		•				
	EN 1650	clean conditions		•			
	EN 13624	clean conditions		•			
		dirty conditions		•			
EN 13697	clean conditions		•				
fungicidal (Aspergillus brasiliensis)	DGHM	clean conditions					•

\*including phase 2 step 1 and phase 2 step 2 tests (quantitative suspension tests and quantitative carrier tests)



# DESCOCEPT AF WIPES

Alcoholic wipes for rapid disinfection

### Composition

100 g solution contain:

Ethanol 42 g,

Didecylmethyl ammonium chloride 0,05 g.

Contains perfumes.

### Material compatibility

Konrad Hornschuch AG, Weißbach

Approval for DESCOCEPT AF on Skai® upholstery material .

Bergé & Co. GmbH, Versmold

Approval for DESCOCEPT AF on MM-Desinfektionsmatten.

### Expert opinions

We have extensive expert reports for all our products. Upon request we are glad to supply you with an expert report portfolio.

### Precautionary and hazard statements

Flammable solid. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Do not breathe vapour. Use only outdoors or in a well-ventilated area. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

For professional use only.

Product	Single unit	Content	Delivery unit	Size	REF	PZN
DESCOCEPT AF WIPES	dispenser box	120 wipes	12	14 x 20 cm	00-312-D120	1898486
	refill	120 wipes	12	14 x 20 cm	00-312-R120	1898581

CE 0482

pH 0 1 2 3 4 5 6 7 8 9 10 11 12 13 14

Use disinfectants safely. Always read label and product information before use.

Dr. Schumacher is certified according to DIN EN 13485, DIN EN ISO9001, DIN EN ISO14001, BS OHSAS 18011 and has a validated environment management system according to EMAS and is a member of IHO, VCI, BAH, DGSV and of the DGKH. [www.schumacher-online.com](http://www.schumacher-online.com)

