

Rapid disinfection

DESCOCEPT AF

Alcoholic rapid disinfection

- Ready for use
- Very good cleaning performance
- With fresh scent



Product description

DESCOCEPT AF is a ready-to-use, alcohol-free, rapid disinfectant based on ethanol. DESCOCEPT AF has a pleasant fresh scent. The product is highly effective against enveloped viruses and possesses excellent cleaning properties. DESCOCEPT AF is suitable for surface disinfection in areas where rapid effectiveness is required.

Range of application

According to Biocidal Products Regulation (BPR): Rapid disinfection and cleaning of alcohol resistant medical inventory and all kinds of surfaces. Especially suitable for inaccessible surfaces.

Acc. to EU Medical Devices Regulation: Rapid disinfection and cleaning of alcohol-resistant, non-invasive medical devices.

Application

To ensure complete wetting apply the undiluted solution evenly on alcohol-resistant surfaces and keep them moist for the required contact time.

Application notes

DESCOCEPT AF can be used for wipe disinfection or spray application.

Spectrum of efficacy and contact times			30 s	1 min	2 min	5 min
Bacteria and yeasts						
application recommendation for surface disinfection: Bactericidal, yeasticidal*	VAH / EN 16615	with mechanical action, dirty conditions			•	
	VAH / EN 13697	without mechanical action, dirty and clean conditions				•
tuberculocidal (M. terrae)	EN 14348	clean conditions		•		
mycobactericidal (M. avium and M. terrae)	EN 14348	clean conditions			•	
Viruses						
active against enveloped viruses	RKI/DVV	incl HBV/HIV/HCV	•			
	EN 14476	dirty and clean conditions	•			
active against noroviruses (MNV)	EN 14476	clean conditions				•
active against rotaviruses	RKI/DVV			•		
Additional test results						
bactericidal	EN 1040		•			
	EN 1276	clean conditions		•		
	EN 13727	clean conditions	•			
		dirty conditions		•		
	EN 13697	clean conditions		•		
yeasticidal (Candida albicans)	EN 1275		•			
	EN 1650	clean conditions		•		
	EN 13624	clean conditions		•		
		dirty conditions		•		
	EN 13697	clean conditions		•		

* including phase 2 step 1 and phase 2 step 2 tests (quantitative suspension tests and quantitative carrier tests)



DESCOCEPT AF

Ready-to-use, alcoholic rapid disinfection

DESCOCEPT AF is particularly suitable for use in non-woven fabric wipe dispenser systems with expert-reviewed disinfection performance and a service life of up to 42 days when used together with the ONE SYSTEM non-woven fabric wipe dispenser system or a service life of up to 28 days when used together with the DESCO/ECO WIPES non-woven fabric wipe dispenser system.

Composition

100 g solution contain:

42 g ethanol, 0.05 g didecyldimethylammonium chloride.

Contains perfumes.

Expert opinions

We have extensive expert reports for all our products. Upon request we are glad to supply you with an expert report portfolio.

Precautionary and hazard statements

Flammable liquid and vapour. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Do not breathe vapour. Use only outdoors or in a well-ventilated area. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

For professional use only.

Product	Single unit	Content	Delivery unit	REF
DESCOCEPT AF	Mace bottle	500 ml	10	00-311-005EXP
	Bottle	1 L	12	00-311-010EXP
	Spray head	1 L	4	00-311-010EXP-SF
	Canister	5 L	3	00-311-050EXP
	Canister	10 L	1	00-311-100EXP

The availability of the products and container sizes depend on a successful national registration.

Use disinfectants safely. Always read label and product information before use.

CE 0482



Dr. Schumacher is certified according to DIN EN 13485, DIN EN ISO9001, DIN EN ISO14001, BS OHSAS 18011 and has a validated environment management system according to EMAS and is a member of IHO, VCI, BAH, DGSV and of the DGKH. www.schumacher-online.com

